

V116 Test
Min passing is 80%

1. What test removes of air from the chamber and load, so the steam can penetrate the load?
 - a. Dart
 - b. Leak
 - c. Spore
 - d. Biological
2. What test checks piping and door seal for leaks?
 - a. Dart
 - b. Leak
 - c. Spore
 - d. Biological
3. What mm/Hg per minute is the pass rate for a leak test over the 10 minute test?
 - a. 5
 - b. 15
 - c. 10
 - d. 2.5
4. What **personal protective equipment (PPE)** is required for safety?
 - a. Face shield
 - b. Gown/Goggles
 - c. Arm protection
 - d. All of the above
5. What is the password to the service menu?
 - a. 2001
 - b. 4819
 - c. password
 - d. current year
6. What valve provides flow through it in only one direction
 - a. Ball
 - b. Swing
 - c. Reverse
 - d. Check

7. What test can be used to check solenoids are working properly
 - a. Screwdriver pull
 - b. Tap it
 - c. Listen for humming sounds
 - d. Call the OEM
8. What device is used to discharge condensate (moisture) and non condensable gases while not permitting the escape of live steam?
 - a. Strainer
 - b. Heat Exchanger
 - c. Vacuum Breaker
 - d. Steam Trap
9. Scenario: You receive work order that the sterilizer air filter is filled with water. What is the suspected malfunctioning component?
 - a. Solenoid S9
 - b. Solenoid S35
 - c. Solenoid S2
 - d. Solenoid S1
10. Scenario: You receive a second work order that the sterilizer is alarming "Too long in Charge". What is the suspected malfunctioning component?
 - a. Solenoid S9
 - b. Solenoid S35
 - c. Solenoid S2
 - d. Solenoid S1
11. What should the jacket gauge read on a normal operation of a sterilizer?
 - a. 20 psi
 - b. 30 psi
 - c. 40 psi
 - d. 50 psi
12. What are some reasons to recall a load?
 - a. Failed Biological
 - b. Open a pack in the OR
 - c. Wet pack
 - d. all of the above